

Operating Instructions for Electronic Wheelchair Scale

- ④ Press the ON key located on front of the indicator.
- ⑤ When the indicator reads Zero, press the **TARE** key, enter the wheelchair weight, then press the **ENTER** key. The indicator will show a negative weight equal to the weight of wheelchair.



- ⑨ For storage, use column release to release column and fold down to locked position. Replace storage pin (see *warning label on scale base*). Store in an up-right position, preferably near a wall for maximum protection against damage.

HINT: If indicator is on when storing, press the ZERO key to zero the indicator which activates the automatic shutoff.

- ⑥ Position patient on scale by pulling wheelchair onto scale. **NEVER PUSH** patient on or off scale. Lock wheelchair wheels.
- ⑦ When weight reading is steady, read the patient's weight.
- ⑧ Remove patient from scale by backing down the ramp.

